Dansk

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Instructions for Use

REF Catalogue number

U.S. Federal law restricts this device to sales by or on the order of a physician

LOT Lot Number

Manufacturer Authorized Representative in the European Community

QTY Quantity

Not made with natural rubber latex

Consult Instructions for Use

Single Use

Caution

English

Intended Use: A heat and moisture condenser with bacterial/viral filter is a device intended to warm and humidify gases breathed in by a patient with a tracheostomy or tracheal tube. The addition of a filter removes microbiological and particulate matter from the gases in the breathing circuit.

Directions for Use:

Connect HCHF device (with or without Flex-Elbow™) securely to the endotracheal tube, tracheostomy tube, face mask and/or breathing circuit (as applicable). Pressure check system to ensure leak free connections. Replace every 24 hours, or more often if needed to prevent accumulation of secretions. Compensation for internal dead space may be necessary.

DO NOT USE AN HCHF DEVICE IN THE FOLLOWING SITUATIONS:

1. In patients who cannot tolerate the additional mechanical dead space of this device.
2. In patients who cannot tolerate the additional airway resistance of this device.
3. In patients with voluminous secretions

Warning: Do not rinse, soak, wash or sterilize. Do not add moisture or use in conjunction with heated water humidifiers or nebulizers. Do not allow saline to enter HCHF device when suctioning patients as resistance may temporarily increase. Replace every 24 hours or more often if needed. Due to increased resistance of the device, the patient must be closely monitored with proper alarms turned on. If any complications are observed, such as mucous plugging, proper airway care must be instituted immediately.

EC REP

GE Medical Systems SCS
283 rue de la Miniere
78530 BUC, FRANCE
P +33.1.30.70.40.40

Vital Signs, Inc. a GE Healthcare Company
11039 E. Lansing Circle
Englewood, CO 80112, USA
P +1.973.956.5300

Made in U.S.A.

©2013 General Electric Company - All rights reserved.
General Electric Company reserves the right to make changes in specifications and features shown herein, or discontinue the product described at any time without notice or obligation.
Contact your GE Representative for the most current information.

GE, GE Monogram, and Vital Signs are trademarks of General Electric Company.
Vital Signs, Inc., a General Electric Company, doing business as GE Healthcare.

This single-use product is not designed or validated to be reused. Reuse may cause a risk of cross-contamination, affect the measurement accuracy, system performance, or cause a malfunction as a result of the product being physically damaged due to cleaning, disinfection, re-sterilization, or reuse.

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